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Product Description

Visalys® Core, developed by Kettenbach, is a dual-curing, fluoride-containing composite used for the fabrication of radiopaque core build-ups and core fillings. The automixing, two-component system is based on a multifunctional acryl composite. BPA or BPA precursors are not used in the manufacturing process of this device. Visalys® Core has a firm consistency with good flow behavior and is available in the following shades: white and dentine. Curing can be controlled as required at any time using optional light-curing.

Indications / Techniques

- Different types of core build-ups
- Luting of root posts

Patient Target Group

Patients being treated in the context of a dental procedure.

Warnings

Do not use with known allergies to methacrylic acid ester, benzotriazole compounds, benzoyl peroxide, camphorquinone, or 4-methoxyphenol.

Allergic reactions to the product cannot be ruled out with sensitive persons.

Sensitization is possible on contact with the skin / mucosa.

Materials containing eugenol could inhibit the curing of composites.

If necessary, pulp areas of vital teeth should be treated using suitable pulp / dentine protection.

Keep out of reach of children.

Cautions

Do not use after expiration date.

Do not swallow the material. If swallowed: Consult a medical doctor if problems arise or persist.

Avoid skin and mucosal contact with uncured material.

Avoid contact with the eyes. If there is accidental contact with the eyes: Rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

Avoid contact with articles of clothing, as the material cannot be removed by dry cleaning.

Safety data sheets can also be found at www.kettenbach.com.

Note

The working times apply for products, which have been stored and processed at a temperature of 23°/74°F and 50 % relative humidity. Higher temperatures shorten and lower temperatures extend these times.

Direct exposure to ambient light accelerates the curing of Visalys® Core.

Recommended Use

Core Build-Up

1. Remove existing fillings and caries.
2. Provide for a dry treatment site using appropriate measures (rubber dam).
3. If the bonding agent requires an additional etching stage, complete etching according to the manufacturer's instructions.
4. Apply the bonding agent according to the manufacturer's instructions.
5. Place a matrix around the tooth if the cavity wall is missing and apply Visalys® Core directly in the cavity. When using a core build-up former, fill it directly using Visalys® Core and place on the tooth. Visalys® Core can be sculpted during the working time.

6. Visalys® Core cures automatically after 5 minutes. The material can optionally be light cured for 20 seconds to accelerate curing. The curing depth with light-curing is 2 mm. The minimum light intensity should be 800 mW/cm². In case of lower light intensity, the light-curing time must be extended accordingly. Place the light as close as possible to the material.

When using AdheSE or AdheSE One F (Ivoclar Vivadent) and Adper Prompt L-Pop (3M ESPE) adhesive, apply the core build-up material Visalys® Core in incremental layers (2 mm), and light-cure.

During curing, lower temperatures, e.g. of the product or working in the anterior region, extend the curing time.

7. Remove the matrix or core build-up former, if necessary.
8. Prepare the core after fully curing.
9. Apply a separating medium on the core surface, e.g. using Vaseline, before fabricating a composite temporary restoration.

Luting Root Posts

For the luting of root posts it is recommended to use Visalys® Core with the 9g/5 ml-automix syringe.

1. Prepare the filled root canal for placing the post.
2. If the bonding agent requires an additional etching stage, complete etching according to the manufacturer's instructions.
3. Prepare the root canal, coronal dentine and, if applicable, the enamel section according to the bonding agent manufacturer's instructions. It is essential to remove any excess from the lumen using a paper tip to prevent blockage of the canal.
4. Condition the surface of the post according to the material manufacturer's instructions. Then avoid contamination of the post surface.
5. Apply Visalys® Core in the canal lumen and, if necessary, on the root post. The tip used should always remain in the material during application. Do not use a lentulo.
6. Immediately afterwards, insert the root post (the working time for Visalys® Core in the root canal is 45 seconds). It is recommended that any excess material is specifically distributed on the occlusal stump surface.
7. To fix the post, the material can be light-cured with a suitable polymerization lamp for 20 seconds.
8. Build up the core as described above.

The coronal section of the tooth can be prepared as a core build-up using Visalys® Core. Use of a matrix is recommended during the build-up due to the flowable consistency.

Application

The pastes should be mixed and dispensed directly from the 9g/5 ml-automix syringe or using an Applyfix 8 dispensing gun DS 24 1:1/2:1 and for the 45g/25 ml-cartridge corresponding mixing tips. Only use the original mixing tips MBT Ø 4.2 mm or MLT Ø 2.5 mm and Intraoral/Endo tips!

The following should be observed when using the 45g/25 ml-cartridge (when using the automix syringe, please start at Point 2):

1. Insert the cartridge in the Applyfix 8 dispensing gun. Ensure that the notches of the cartridge base face downwards. If the cartridge is fitted incorrectly in the dispensing gun, the locking clip will not close.
2. Remove the cartridge/syringe cap and, before fitting a mixing tip for the first time, extrude a small amount of material until both materials are delivered uniformly. Then discard the extruded paste.
3. Fit the mixing tip. Turn the cap 90° in a clockwise direction until it engages. If applicable, mount an Intraoral/Endo tip locking on the mixer.

4. Apply constant pressure while extruding material through the mixing tip. The material flow is interrupted as soon as the pressure is eased on the dispenser grip or syringe plunger. Do not use force to extrude material that has already cured in the mixing tips, as the cartridge/syringe and mixing tips could be damaged by creating excess back pressure.
5. Dispose of used mixing tip, replace the cartridge/syringe cap and check the outlet at the start of the next application.

Cleaning and Disinfection of Applyfix 8

Remove the cartridge.

Remove uncured paste using a cloth soaked in alcohol.

The Applyfix 8 dispensing gun can be used several times if properly disinfected. Kettenbach recommends exchanging the gun if there are visible traces of use. Disinfection can be carried out using a commercially available dipping disinfectant. Kettenbach recommends the use of glutaraldehyde-based solutions.

Storage Conditions

Store dry and protect from sunlight, storage in a refrigerator or at room temperature (2°C/36°F–24°C/75°F), avoid extreme temperatures. The product must be heated to room temperature before use when it has been stored in a refrigerator.

Constituents

Main components of Visalys® Core are approx. 42 vol% inorganic fillers in the size range 0.2–20 µm including ytterbium fluoride.

The polymer base consists of aliphatic dimethacrylates.

Classification

ISO 4049, Type 2, Class 3.

Disposal

The empty automix syringes/cartridges/mixing tips/appliator tips and cured material can be disposed in the household rubbish.

Trademarks

Visalys® Core is a registered trademark of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials/products are available in certain markets only.

For dental professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the legal theory asserted including warranty, contract, negligence or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply and consider all recommendations and information in connection with any use.

Updated: August 31, 2023

Technical Data

Working and curing times

Working time	90 seconds
Working time in the root canal	45 seconds
Light-curing (in a light wavelength range 400-500 nm)	20 seconds
Self-curing	5 minutes
Radiopacity*	2.5 mm Al
Flexural strength	> 100 MPa

Color stability meets the requirements of ISO 4049.

*Aluminum has a radiopacity equivalent to dentin. Therefore, 1 mm of a material that has a radiopacity equivalent to 1 mm of aluminum has a radiopacity equivalent to 1 mm of dentin, and 2 mm of aluminum is equivalent to the tooth enamel.

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